

Veterinary Rapid Tests

OUR PRODUCTS

One Step Rapid
Test for Veterinary
Let him grow up
healthy and happy



With the improvement of national living standards, pet owners will invest more in animal health care. With the improvement of the scale of livestock and poultry breeding industry and the change of the situation of animal epidemic prevention and control, the demand for large-scale animal detection is also increasing.

HOW TO USE?



Tilt the cotton swab 45 ° into the anus or collect fresh feces for sampling

Note: the amount of feces should not be collected too much to prevent blocking from affecting the test results


Insert the collected sample swab vertically into the bottom of the sample extraction tube with buffer, rotate it in the solution 10 times

Put on the dripper

Add 2 to 3 drops of the sample without bubbles (please vertical and slowly drop by drop) into the sample hole(s)

Read results at 5-10 minutes

INTERPRETATION OF RESULTS

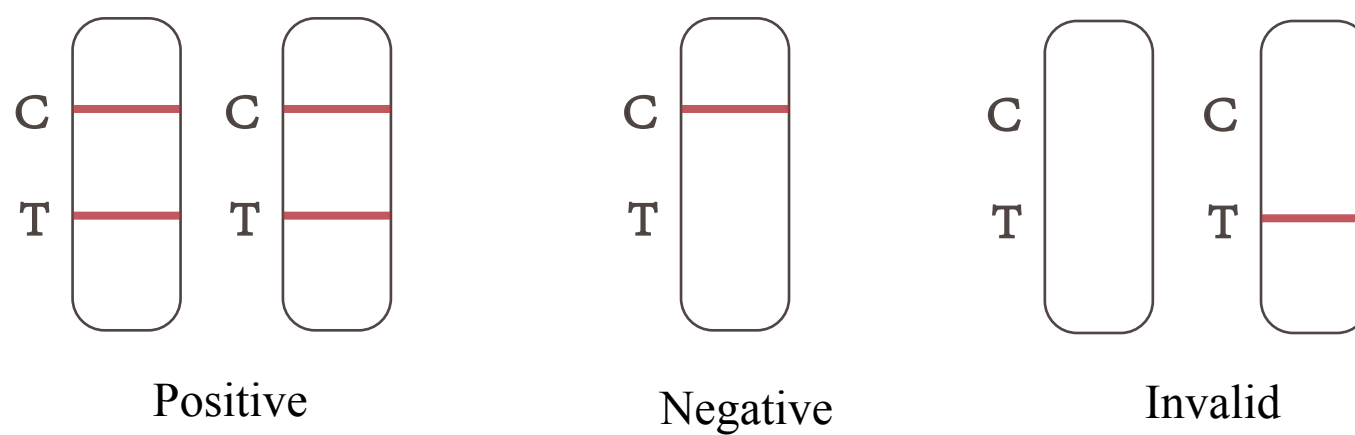

Positive

Negative

Invalid

Items	Specimen	Format	Pack
Canine Parvovirus Antigen (CPV Ag)	Feces/Vomit	Cassette	10 T
Canine Coronavirus Antigen (CCV Ag)	Feces/Vomit	Cassette	10 T
CPV + CCV Ag Combo	Feces/Vomit	Cassette	10 T
Canine Influenza Virus Antigen (CIV Ag)	Secretions	Cassette	10 T
Canine Distemper Virus Antigen (CDV Ag)	Secretions	Cassette	10 T
Canine Adenovirus Type - II Antigen (CAV Ag)	Secretions	Cassette	10 T
CDV+CAV-II+CIV Antigen Combo	Secretions	Cassette	10 T
Feline Panleucopenia Virus Antigen (FPV Ag)	Feces/Vomit	Cassette	10 T
FCoV Antigen (FCoV Ag)	Ascites/Hydrothorax	Cassette	10 T
FCoV+FPV Antigen Combo	Feces	Cassette	10 T
Feline Herpes Virus Antigen (FHV Ag)	Eye Secretion	Cassette	10 T
Feline Calicivirus Antigen (FCV Ag)	Secretions	Cassette	10 T
Canine/Feline Toxoplasma Antigen (TOXO Ag)	Feces	Cassette	10 T

Web: www.prometheusbio.com

E-mail: sales@prometheusbio.com

Address: 2nd floor, No. 3, 8th Street, Qiantang Area, 310018 Hangzhou, Zhejiang, China